

PUBLIC HEALTH WALES—LTBI Request Form

Telephone: 020 8266 6510



NHS NUMBER

--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

SURNAME

FORENAME

DATE OF BIRTH (DD/MM/YYYY)

--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

PATIENT ADDRESS

REQUESTING CONSULTANT/TB NURSE

DEPARTMENT

HOSPITAL

GENDER

M	<input type="checkbox"/>	F	<input type="checkbox"/>
---	--------------------------	---	--------------------------

HOSPITAL NUMBER

--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

EPISODE / ACCESSION NUMBER

Place Barcode In This Box

CLINICAL DETAILS

IMMUNOSUPPRESSIVE THERAPY

Y	<input type="checkbox"/>	N	<input type="checkbox"/>
---	--------------------------	---	--------------------------

Referring Microbiology Laboratory (Please tick location as appropriate)

ABERYSTWYTH: Bronglais General Hospital

Location Code: **PHWMA**

LLANTRISANT: Royal Glamorgan Hospital

Location Code: **PHWRGH**

BANGOR: Ysbyty Gwynedd

Location Code: **PHWMN2**

NEWPORT: Royal Gwent Hospital

Location Code: **PHWRH1**

CARDIFF: University Hospital Llandough

Location Code: **PHWMC**

RHYL: Ysbyty Glan Clwyd

Location Code: **PHWMN1**

GLANGWILLI: West Wales General Hospital

Location Code: **PHWMC1**

SWANSEA: Singleton Hospital

Location Code: **PHWSH**

HAVERFORDWEST: Worthybush General Hospital

Location Code: **PHWWGH**

WREXHAM: Ysbyty Maelor

Location Code: **PHWMN3**

REFER TO WWW.SWL.PATH.NHS.UK FOR SAMPLE TUBE AND SAMPLE COLLECTION REQUIREMENTS

PUBLIC HEALTH WALES—LTBI Request Form

LABORATORY INFORMATION:

SAMPLE COLLECTION DATE:	(DD/MM/YYYY)	TIME:	(HH:MM)
	<input type="text"/>		<input type="text"/>
INCUBATION OF QFT-PLUS TUBES			
TIME IN:	<input type="text"/>	DATE:	<input type="text"/>
	<input type="text"/>		<input type="text"/>
NAME (Print):	<input type="text"/>		
	<input type="text"/>		
TIME OUT:	<input type="text"/>	DATE:	<input type="text"/>
	<input type="text"/>		<input type="text"/>
NAME (Print):	<input type="text"/>		
	<input type="text"/>		
QFT-PLUS TUBES CENTRIFUGED ON	DATE:	<input type="text"/>	

GUIDANCE NOTES:

1. Clearly record date and time sample was taken and tick the location of the requesting laboratory.
2. The sample must be packed with this form for transport to SWLP.
3. Please ensure the QFT[®]-Plus blood tubes have been centrifuged at 3,000 g for 15 minutes within 3 days post incubation.
4. QFT[®]-Plus tubes can then be stored upright at 2-8°C for up to 28 days following centrifugation.
5. The sample is to be transported via DX courier service.

REFER TO WWW.SWLPATH.NHS.UK FOR SAMPLE TUBE AND SAMPLE COLLECTION REQUIREMENTS