
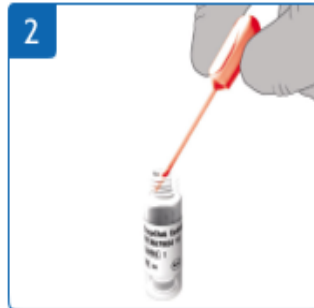


Roche CoaguChek Pro II Quality Control Testing Guide	Q Pulse Reference Number POC-PINS-16	 MAKING EVERY TEST MATTER
Version Number 1.0	Author Cecilia Anghelescu	


## CoaguChek Pro II Quality Control Testing



1 Remove plastic cap and rubber bung.



2 Ensure no liquid is in the end of pipette and carefully cut end off and empty all contents into glass vial, ensuring not to touch the powder. Exercise normal precautions when handling laboratory reagents.

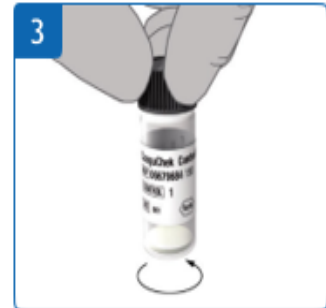

3 Replace the black cap and swirl on desktop (do not shake), until all the powder has dissolved. The solution is now stable for 30 minutes.




4 Touch 'Control Test'.




5 Insert test strip when prompted and test strip code chip (If prompted).



6 Check lot number on side of QC bottle then select from list, or select 'New'.

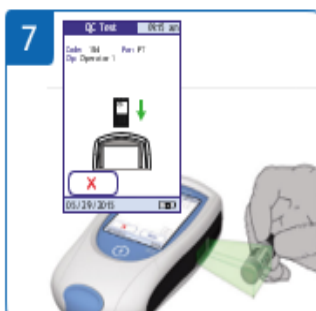

7 If 'New' option selected, insert QC code chip when prompted or select 'Scan' and scan the side of the QC bottle using the barcode.

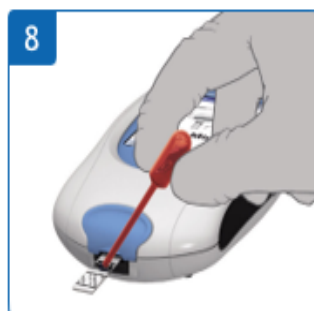

8 Once the 3-minute countdown starts, apply the solution to the test strip using the pipette.



9 The sample will be analysed and the result and acceptable range displayed on the screen.