

# Werfen GEM Hemochron 100 analyser Patient User Guide

## 1. Power On and Select Go




Press blue power button on instrument (1).  
Tap screen and select **GO!** (2).


## 2. Insert Cartridge




Insert cartridge when prompted. (3)


## 3. Enter OID/PID (optional)

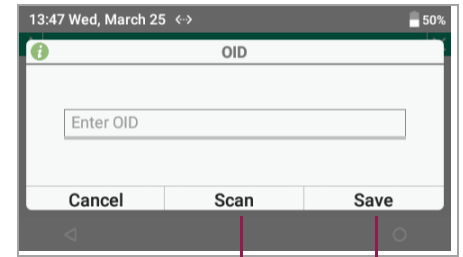

If prompted, enter or scan Operator ID or Patient ID. Select **Scan** (4) and scan when  displayed.


Select **Save** (5).  
Wait while  (warming cartridge) is displayed.

## 4. Add Sample

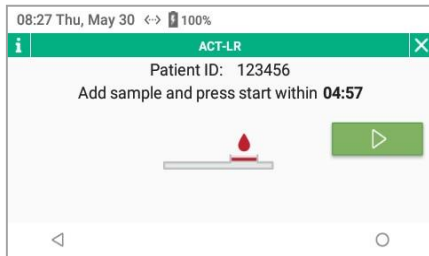


Add sample into cartridge sample well (6).






Select  (7). Instrument will count up and display result after clotting detected.
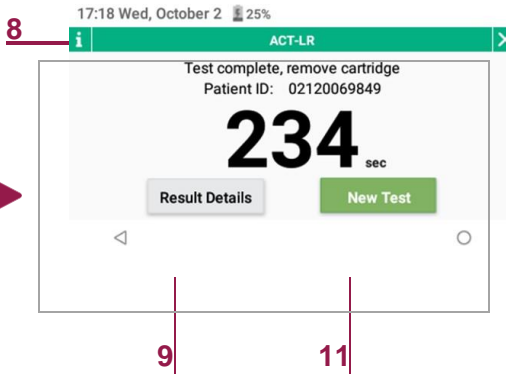
## 5. Display Results



Results display indicates test is complete. Remove cartridge and dispose.

### Optional

- Select  (8) to access ACT patient-test history
- Select  (9) to review test information or add note (10)
- Select  (11) to start new patient test

**Note:** Test result will be stored in Database  and sent to Laboratory Information System (LIS), if auto-send is on.

