


Roche Cobas B101 Lipid Quality Control Guide	Q Pulse Reference Number POC-PINS-18	 MAKING EVERY TEST MATTER
Version Number 1.0	Author Cecilia Angheliescu	

## Roche Cobas B101 Lipid test Quality Control Guide

### Step



- 1 Material needed:
- A) 1 Lipid QC Info disc
  - B) 1 QC solution level 1
  - C) 1 QC solution level 2
  - D) 2 Lipid discs



- 2 Power on the instrument.



- 3 Choose **Control Test** and then select **QC Test**.



- 4 For a new QC lot, choose **New QC Lot**. To select a previously tested QC lot, choose a lot button.



- 5 For a new QC lot, take the Lipid QC Info disc from the QC solution kit.



- 6 To open the lid, choose **Open** on the screen, then follow the on-screen instructions.




- 7 Place the QC Info disc on the turntable within the white circle.












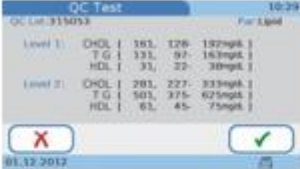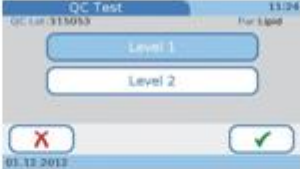

- 8 Close the lid.

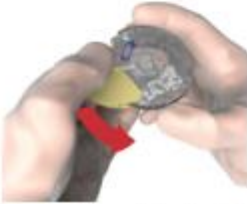C

Roche Cobas B101 Lipid Quality Control Guide	Q Pulse Reference Number POC-PINS-18	 MAKING EVERY TEST MATTER
Version Number 1.0	Author Cecilia Anghelescu	

## Roche Cobas B101 Lipid test Quality Control Guide

Step		
		
<p>9 The target values and target ranges for level 1 and 2 are displayed. To validate the values, choose . The QC information is saved on the system.</p>	<p>10 Choose the level of the QC test (Level 1 or Level 2).</p>	<p>11 Open the disc pouch and remove the Lipid disc.</p>
		
<p>12 Turn the disc on its back and open it completely.</p>	<p>13 Carefully apply a drop of QC solution to the suction point. Make sure it is of the level that you have just chosen.</p>	<p>14 Check the sample volume: the area marked in blue must be completely filled with QC solution.</p>
		
<p>15 Close the hinge cover firmly. Once closed correctly, the hinge cover cannot be opened again.</p>	<p>16 Open the lid. Remove the Lipid QC Info disc. Insert the Lipid disc. Close the lid.</p>	<p>17 Verify that the QC solution lot number matches the lot number displayed on the screen. To begin the measurement, choose .</p>