
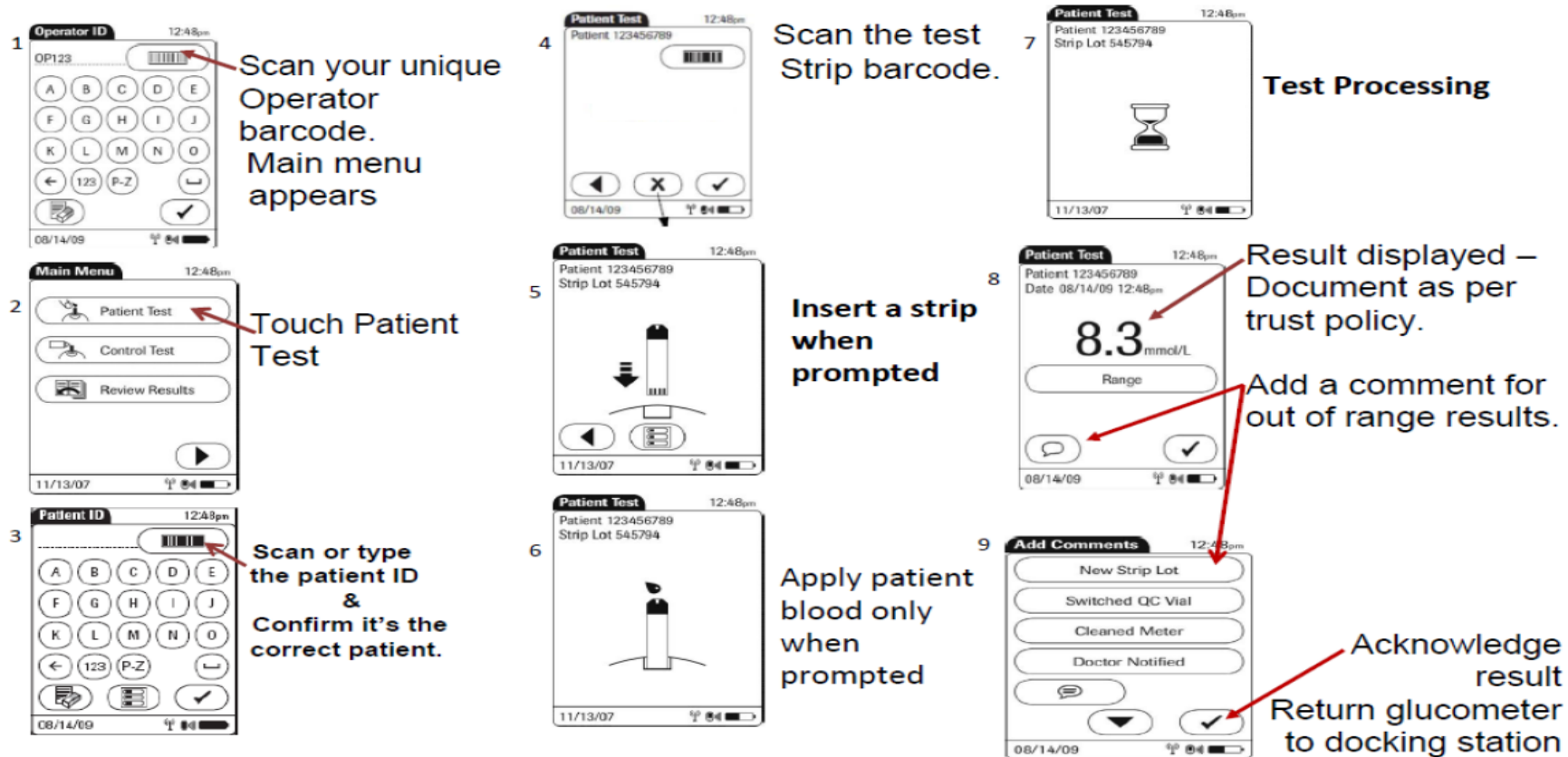


Title of Document QUICK GUIDE: HOW TO PERFORM A PATIENT BLOOD GLUCOSE TEST			 MAKING EVERY TEST MATTER
Version Number 2.0	Q Pulse Reference Number POC-PINS-3	Author Faye Browne	

QUICK GUIDE: HOW TO PERFORM A PATIENT BLOOD GLUCOSE TEST

ANY ISSUES PLEASE EMAIL ON SITE POCT TEAM MONDAY TO FRIDAY 9AM TO 5PM AS BELOW

STG (POCT@stgeorges.nhs.uk), KH (khft.poct@nhs.net), CUH (Ch-tr.cuhpoct@nhs.net), RNOH (rnoh.poct@nhs.net)



1 Scan your unique Operator barcode. Main menu appears

2 Touch Patient Test

3 Scan or type the patient ID & Confirm it's the correct patient.

4 Scan the test Strip barcode.

5 Insert a strip when prompted

6 Apply patient blood only when prompted

7 Test Processing

8 Result displayed – Document as per trust policy. Add a comment for out of range results.

9 Acknowledge result Return glucometer to docking station